

Features 功能



No need for manual suction

無需手動吸入

Battery-operated device gently removes congestion

電池運作可避免分泌物阻塞



2 gentle suction speeds

2 檔吸力

Start with the lower suction setting and increase if needed

開始設定為低檔，亦可增加吸力



2 sizes of nasal tips

2 種尺寸的吸嘴

For infants and toddlers

適合嬰兒及兒童



Quiet & comfortable

寧靜舒適

For stress-relief from nasal congestion

舒緩鼻塞帶來的壓力



Dishwasher safe parts

部件可放入洗碗機

Making it easier for you to clean after each use

每次使用後更容易清潔

Certain trademarks used under license from The Procter & Gamble Company or its affiliates.

部份商標為寶潔公司或其關聯公司認可使用。

© 2019 All rights reserved.

奧利佳(香港)有限公司

香港新界葵涌大連排道202-210號偉倫中心一期11樓

電郵: braunhealthcare@dchauriga.com

電話: 9869 3782

2019LFHKV01
JUL 2019

BRAUN

Designed for what matters



NEW

Nasal aspirator 1

電動吸鼻器

Clears stuffy noses fast & gently.

It's the little things that matter most.

迅速、溫和


清理鼻腔分泌物

小小工具 大大用處




BNA100


How to use 如何使用

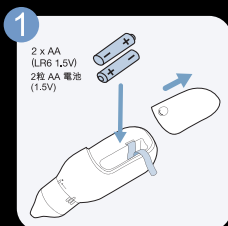

Insert batteries.
裝上電池。

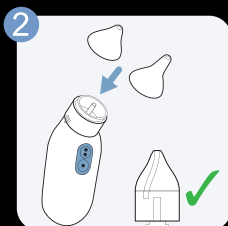


Select tip & press in place.
選擇吸嘴並將其套上。

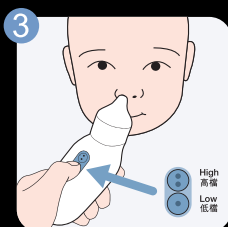

Insert the tip gently into one nostril, securely but comfortably.
以安全舒適的方式，輕輕把吸嘴插入其中一個鼻孔。



Choose between two suction settings.
從兩檔吸力之中選擇其一。

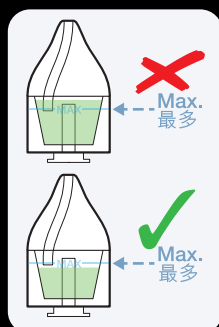


Do not overfill reservoir.
切勿溢出收集杯。


Cleaning Instructions 清潔指南




Twist to unlock reservoir.
扭動以擰開收集杯。

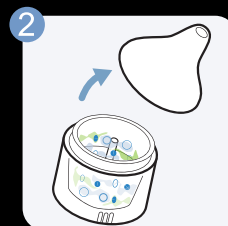


Gently pull off tip.
輕輕取下吸嘴。

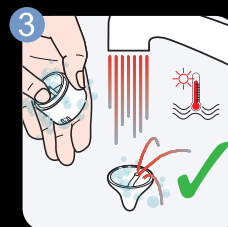

Rinse/dishwash reservoir & tips.
沖洗收集杯及吸嘴。

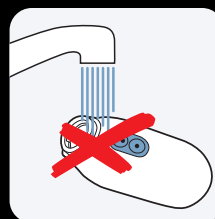

Do not rinse/dishwash device. Wipe to clean.
切勿沖洗本電動吸鼻器。
輕輕擦拭表面。

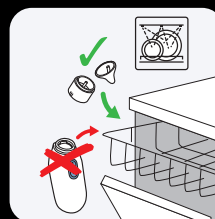

Available to put detachable parts (reservoir & tips) in the dishwasher.
亦可把拆卸部件(收集杯及吸嘴)放入洗碗機清潔。

溫馨提示：水溫不得超過100℃或蒸汽溫度。如溫度超出100℃，透明的收集杯有機會出現變形。

Note: Water temperature cannot exceed 100°C or steam temperature. If temperature exceeds 100°C, the transparent reservoir may deform.